

## Adper™ Prompt™ Self-Etch Adhesive

### Preparation:

- Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry. Note: Do not use any cleaning medium that contains oils or fluoride.
- Prepare adhesive according to dispensing instructions (see opposite side).

- Brush adhesive onto surface to be sealed; massage for 15 seconds applying moderate pressure. **Do not shorten this time. Proper bonding will not occur if the solution is simply applied and dispersed.**



- Use a gentle stream of air to **thoroughly dry** the adhesive to a thin film. A second layer is not required for bonding sealants.



- Apply a **light-cured** sealant. (Note: Adper Prompt self-etch adhesive is not indicated for self-cure sealants)



- Light-cure the sealant and adhesive together with a halogen, LED or plasma light for the amount of time specified to cure the sealant.



## Adper™ Prompt™ Self-Etch Adhesive

**Dispensing:**

- Hold vial in a vertical orientation as shown.
- Dispense one drop from **solution A** into the disposable mixing well.
- **Note:** holding the vial vertically for a few seconds before dispensing allows trapped air to escape out of the vial tip.



- Hold vial in a vertical orientation as shown.
- Dispense one drop from **solution B** into the disposable mixing well.
- **Note: to ensure optimum drop ratios, keep the vials vertical when dispensing.**



- Mix the solution aggressively for 5 seconds with the disposable mixing brush.

*2 of 2*