

Adper™ Scotchbond™ Multi-Purpose Plus Adhesive System

Preparation:

- Prepare a standard amalgam preparation. Roughen any existing amalgam and follow the same bonding procedure outlined below for enamel and dentin.
- When bonding amalgam rub the inner surface of the matrix band with wax to prevent inadvertent bonding.
- Place matrix.

Etchant:

- Apply Scotchbond™ Etchant to enamel and dentin—wait 15 seconds.
- Rinse.
- Remove excess water with an air syringe or by blotting. Appearance should be shiny. If surface has matte appearance it should be remoistened.



Activator/Primer:

- Mix one drop each of Activator and Primer.
- Apply to etched enamel and dentin — wait 15 seconds.
- Dry gently for 5 seconds.



Adhesive Application:

- Mix one drop each of Adhesive and Catalyst.
- Apply the mixed adhesive to the primed enamel and dentin.
- Thin the mixture using a brush tip, if desired. Avoid excessive air thinning.



**Adper™ Scotchbond™ Multi-Purpose Plus
Adhesive System**



4

- Triturate amalgam.



5

Completing the Restoration:

- Condense and burnish the amalgam.