General Information

3M™ ESPE™ RelyX™ Veneer Cement is a radiopaque, light-cured resin cement indicated for bonding veneers fabricated of porcelain or composite. It is used in conjunction with 3M™ ESPE™ RelyX™ Try-In Pastes, 3M™ ESPE™ Single Bond Dental Adhesive System or 3M™ ESPE™ Scotchbond™ Multi-Purpose Dental Adhesive System and 3M™ ESPE™ RelyX™ Ceramic Primer. The RelyX Veneer cement contains a resin system consisting of a TEGDMA/BisGMA blend and a zirconia/silica filler with an average particle size of 0.6 microns.

Precautions

RelyX Veneer cement contains acrylate resins. Avoid use of this product on patients with known acrylate allergies. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured resins. **Use of protective gloves and a no-touch technique is recommended.** If skin contact occurs, wash skin with soap and water. Acrylates may penetrate commonly-used gloves. If cement contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If irritation persists, consult a physician.

Instructions for Use

RelyX Veneer cement is designed to be used in conjunction with Single Bond or Scotchbond Multi-Purpose dental adhesive systems and RelyX Try-In pastes. It is recommended that RelyX Veneer cement be used only with 3M ESPE adhesive systems where veneer bonding is indicated.

Note: 3M ESPE disclaims liability for claims arising from the use of RelyX Veneer cement with any adhesive other than Single Bond dental adhesive and Scotchbond Multi-Purpose dental adhesive.

1. To prepare the veneer, the internal bonding surface of a ceramic veneer surface should be microetched with a hydrofluoric acid etchant. The dental laboratory typically does this. The internal surface of a composite veneer should be roughened by sandblasting or air abrasion.
2. Remove the provisional restoration, if present, from the tooth. Clean tooth with pumice; rinse thoroughly and lightly air dry.
3. Check the fit and esthetics of the restoration by trying in the veneer with water or the Translucent shade of RelyX Try-In paste. If the fit and esthetics are acceptable, proceed directly to step 5 using the Translucent shade of RelyX Veneer cement.
4. If color adjustment is needed, select the appropriate shade of RelyX Try-In paste. Apply RelyX Try-In paste directly onto the internal surface of the veneer. Gently seat the veneer onto the tooth preparation. With multiple veneers, start at the midline and seat veneers sequentially working back. The shade of RelyX Try-In paste is formulated to precisely match the corresponding cured shade of the cement.

Once the fit and esthetics are verified, remove the veneers and thoroughly rinse the RelyX Try-In paste from the tooth and restoration with a water spray and then dry.

For Single Bond dental adhesive: Dispense 2-3 drops of Single Bond dental adhesive into the disposable mix well. **Do not allow solvent to evaporate from the adhesive prior to placement.** Using a fully saturated brush tip for each coat, apply 2 consecutive coats of the adhesive to the etched dentin and enamel. Dry gently for 2-5 seconds to evaporate the solvent. **Do not light cure.** Apply 1 coat of Single Bond dental adhesive to the silane treated veneer. Dry gently for 2-5 seconds. **Do not light cure.**

For Scotchbond Multi-Purpose dental adhesive: Apply Scotchbond Multi-Purpose Primer to etched enamel and dentin. Dry gently for 5 seconds. Apply Scotchbond Multi-Purpose Adhesive to the primed enamel and dentin. **Do not light cure.** Apply 1 coat Scotchbond Multi-Purpose Adhesive to the silane treated veneer. **Do not light cure.**

9. Apply a thin layer of the selected shade of RelyX Veneer cement directly from the syringe onto the bonding surface of the veneer. **Protect the cement from direct operatory light exposure.**

10. Seat the veneer in place using gentle pressure. Excess cement will extrude out around the margins. Spot cure the veneer into place on the facial surface away from the margins using a small diameter light guide for 20 seconds to secure it in place.

11. Remove the excess cement from the margins using a blunt instrument or dry brush. Alternatively, the excess cement can be "tack-cured" for 5 seconds with a **standard halogen light** to allow the excess to gel and allow for easier cleanup.

12. Light cure the labial, lingual, interproximal and occlusal surfaces for 30 seconds each. The A5/Dark shade will require a 40 second exposure for each surface. **Note: The cure times are indicated for a standard halogen curing light with a minimum light output of 400 mW/cm².**

13. Remove matrix strips. Finish marginal areas with 3M™ ESPE™ Sof-Lex™ Finishing and Polishing Strips and Discs. Check and adjust occlusion as necessary.

Storage and Use

1. RelyX Veneer cement is designed to be used at room temperature of 21-24°C or 70-75°F.

2. Shelf life in a refrigerator or at room temperature is 24 months. See outer package for expiration date. Storage in refrigerator ensures longest possible shelf life. Allow cement to come to room temperature before use.

3. Do not store in proximity to eugenol-containing products.

4. Do not expose the cement to elevated temperatures or intense light.

Warranty

3M ESPE will replace product that is proven to be defective. 3M ESPE does not accept liability for any loss or damage, direct or consequential, arising out of the use or the inability to use these products. Before using, the user shall determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.

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Cemento Para Carilla

5. The internal surface of the veneer should be thoroughly cleaned prior to applying the silane to remove any contaminants. The surface can be cleaned by applying 3M™ ESPE™ Scotchbond™ Etchant (35% phosphoric acid) for 15 seconds followed by rinsing and drying. Dispense RelyX Ceramic Primer (silane) into a mixing well. Brush a layer of the silane onto the internal surface of the veneer. Lightly air-dry the surface for 5 seconds to evaporate the solvent.

6. Clean the prepared teeth using flour of pumice slurry. Rinse and lightly dry. Isolate the area to prevent contamination. Place matrix strips between teeth to prevent unwanted bonding to adjacent teeth.

7. Etch the tooth preparation. Apply Scotchbond etchant to both dentin and enamel for 15 seconds. Rinse for 10 seconds. **Blot excess water leaving tooth moist. If the tooth is accidentally dried, rewet the bonding surface with water for 15 seconds and blot excess leaving tooth moist.** Alternatively, a HEMA based desensitizing agent can be applied to the etched dentin according to the manufacturer's instructions.

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© 3M 2001 44-0023-3831-5-A