# **3M** Dispenser, type HP

**Dispenser, type HP**

**Pistolet distributeur - type HP**

**Dispenser, typ HP**

**Dispenser, tipo HP**

**Dispensador, tipo HP**

**Dispenser, type HP**

**Blandningspistol, typ HP**

**Jakelija, malli HP**

**Dispenser, τύπου HP**

**Pistola, tipo HP**

**Aftrykspistol, type HP**

Instructions for Use

Mode d'emploi

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Bruksanvisning

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
5/99

## ENGLISH


The 3M Dispenser, type HP simultaneously mixes and delivers the impression material directly from the cartridge.

Push the release lever up on the 3M Dispenser (a) and fully retract the plunger (b).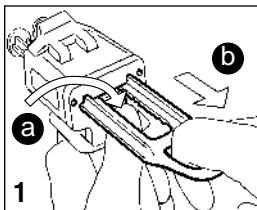


### To load the cartridge:

1. Place the cartridge lock in the up position (a).
2. Determine the proper cartridge orientation by locating the "v-shaped" notch on the flange of the cartridge.
3. Insert the cartridge flange directly behind the retainers on the dispenser, ensuring that the "v-shaped" notch is **facing down**.
4. Place the cartridge lock in the down position.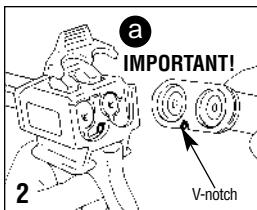


### To remove the cap:

Remove the cartridge cap by rotating the cap 90 degrees in a counter-clockwise direction. The cap is now in the "unlock" position.

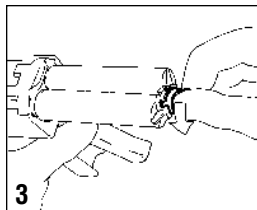
Next, place the thumb on the top of the cartridge cap and the index finger underneath the cap. With **firm** pressure tilt the cap downward and peel the cap away from the cartridge.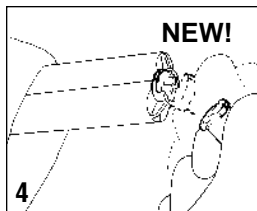


**NOTE: Do not dispose** of the cartridge cap. To store the cartridge after use, remove the used mix tip and replace the cartridge cap.

Gently squeeze the dispenser to extrude enough material from the cartridge to ensure even flow of both base and catalyst. If this does not occur, check for any plugged material and remove with an explorer. Wipe the end of the cartridge clean.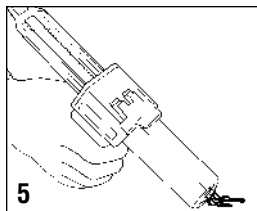


### To attach the mix tip:

Align the “v-shaped” notch on the rim of the mix tip to the “v-shaped” notch located between the cartridge barrels. Push firmly to attach the mix tip to the cartridge.

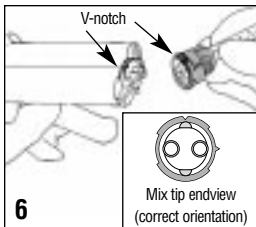
Next, rotate the **colored collar** of the mix tip 90 degrees in a clockwise direction to its “locked” position. Dispenser is now ready for use.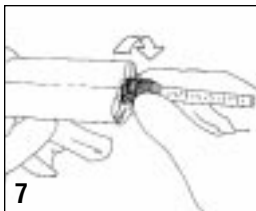


### Cartridge storage/ mix tip removal:

1. Rotate the collar on the mix tip 90 degrees in a counter-clockwise direction to the “unlock” position.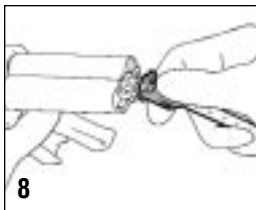

2. Place the thumb on top of the mix tip with the index finger below.  
3. Tilt the mix tip downward and peel it away from the cartridge.


Next, place the cap on the used cartridge (a), and turn the cap to its locked position (b). Alternately, the used mix tip may remain in place as a storage cap until the next use.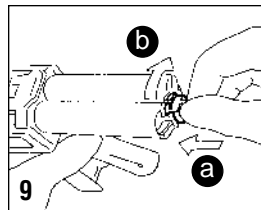


**NOTE:** At next use, dispense enough material to ensure that both base and catalyst flow evenly before placing a new mix tip on the cartridge.

### To remove the cartridge:

Push the dispenser release lever up (a). Fully retract the plunger (b).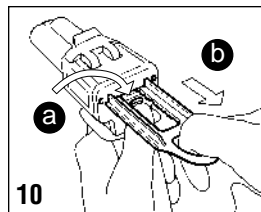

Next, place the cartridge lock in the up position (a) and lift the cartridge out of the dispenser (b).

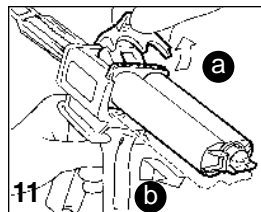## Important Information and Troubleshooting:

**Mix Tip:** Align one of the two posts on the inside of the mix tip collar with the “v-shaped” notch on the rim of the collar (see figure 6). If the post and “v-shaped” notch are not aligned it will not be possible to push the mix tip flush with the impression material cartridge, or to turn the collar to its locked position.

**Plunger:** Squeeze the dispenser handle to engage the plunger into the cartridge. If the plunger does not enter the cartridge, check to see that 1) the cartridge and plunger are properly aligned and 2) the cartridge has been inserted in the proper orientation (see figure 2).

**Trigger Stroke:** Located at the interface between the dispenser trigger and its housing is a red, removable wedge. For use with smaller hands, keep the wedge in place. For use with larger hands, the wedge may be removed (by sliding forward) for greater trigger strokes.

**Cleaning the Dispenser:** Clean the dispenser as needed. The dispenser is detergent safe and requires no lubrication. The dispenser may be disinfected by using liquid sterilants or may be sterilized using a steam autoclave. Do not sterilize using dry heat or chemical vapor as damage to the dispenser may result.

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© 3M 1999 44-0019-9025-6-B