# 3M ESPE

## Restorative Dispenser

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## ENGLISH

### General Information

The 3M™ ESPE™ Restorative Dispenser is a device used to dispense 3M ESPE capsule materials.

### Indications

Dispensing of 3M ESPE dental materials contained in capsules.

### Precautionary Information for Dental Personnel

- Seat capsules securely in dispenser barrel and engage capsule as directed.
- Do not use excessive dispensing force. Excessive force may cause the capsule to dislodge and may result in patient injury.
- This product is recommended to be used with 3M ESPE capsules.
- Follow all instructions for proper use. Any deviation from the instructions for use shall be at the discretion and the sole responsibility of the dental practitioner.

### Instructions for Use

#### Directions

1. Load the capsule into the top opening of the dispenser barrel as shown in figure 1. Pull the capsule forward manually and/or by squeezing the handle so that the plunger pushes the capsule forward into the engaged position. The engaged position is shown in figure 2.

figure 1

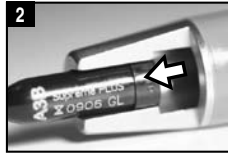
figure 2

2. With the plunger and capsule lightly engaged, rotate the capsule to the desired angle.
3. Dispense paste by applying slow, steady pressure to dispenser handle.
4. Lift the handle to disengage the plunger and lift the capsule out of the dispenser barrel.

### Storage and Use

**Sterilization Recommendations.** Raise handle up and remove plunger. Thoroughly remove any material buildup from the dispenser prior to sterilizing.

Listed below are four different steam sterilization cycle types that will effectively sterilize the dispenser:

- 15 minutes at 121°C/250°F wrapped
- 10 minutes at 121°C/250°F unwrapped
- 10 minutes at 132°C/270°F wrapped
- 3 minutes at 132°C/270°F unwrapped

Do not use chemi-clave. Do not use temperatures exceeding 140°C/285°F. Repeated exposure to high temperature can degrade plastic. Discard product should deterioration in product appearance or performance occur.

The sterilization instructions above have been validated by 3M ESPE as being capable of preparing this device for multiple uses. It is the responsibility of the dental professional to ensure that desired results are achieved and the process is conducted using proper equipment, materials and trained personnel. This requires validation and routine monitoring of the process. Any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences.

### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Caution:** US Federal Law restricts this device to sale or use on the order of a dental professional.

### Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

### Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M and ESPE are trademarks of 3M or 3M ESPE AG.
© 3M 2009. All rights reserved. 44-0007-4809-3
**3M ESPE AG**