ENGLISH

GENERAL INFORMATION

Concise™ composite is a chemically activated, strong, esthetic long-lasting anterior restorative material. Retention and sealing are provided through the use of the acid etch technique and an adhesive system.

Concise composite is recommended for use in Class III and Class V restorations and for limited use in Class I restorations in premolars and selected Class IV restorations where esthetics are of primary importance.

Concise composite contains 65% (by volume) quartz filler with an average particle size of 9 microns (the particle size ranges from 1 to 40 microns).

PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS

Concise composite contains BIS-GMA and TEGDMA. Avoid use of this product on patients with known acrylate allergies. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resins should be avoided. If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If skin contact occurs, wash skin with soap and water.

INSTRUCTIONS

1. Prophyl:

Prophy the enamel surfaces to be etched with a pumice-water slurry. Do not use prophyl paste containing oil or fluoride. Rinse thoroughly.

2. Isolation:

Isolate the area to be restored by use of rubber dam or cotton rolls. The patient is not allowed to rinse until after the composite placement.

3. Tooth Preparation:

Remove carious lesion and/or old restoration to sound dentin or enamel.

4. Pulp Protection:

In deep preparations, the pulpal floor should be coated with a hard-setting calcium hydroxide base restricting it to the DEEPER portion of the cavity, leaving dentin and enamel walls free for bonding.

5. Etching Enamel:

Dry adjacent enamel surfaces and cavity. Acid etch all of the enamel surfaces using appropriate dental acid etching materials following the manufacturer's directions for time.

Thoroughly rinse etched areas. Do not allow the patient to rinse or contaminate the etched areas.

Dry enamel and dentin with oil and moisture-free air. The etched surfaces should be chalky white. If not, repeat etching.

6. Adhesive Placement:

Apply the bonding agent following the manufacturer's instructions.

7. Concise Composite Dispensing and Mixing:

Use one end of the disposable mixing spatula to place an appropriate volume of paste A (approximately half the size of the restoration) on the mixing pad. Use the other end of the spatula to place approximately the same volume of paste B on the pad.

Mix the pastes together thoroughly for at least 20 seconds. Properly mixed material should feel and look evenly blended. If an agate spatula is preferred, take extra precaution not to cross-contaminate the contents of the jars. Metal spatulas will discolor the mix and are not recommended.

The working time of Concise restorative material can be adjusted to the dentist's needs by altering the ratio of the two pastes. Ratios up to 2 : 1 can be used. To extend working time, use more paste B than paste A. To shorten, use more paste A than paste B.

8. Concise Composite Placement:

The working time for the materials from start of the mix is a minimum of 90 seconds (23°C or 72°F).

Place the mixed pastes using a non-metallic placement instrument or syringe. The material is easily worked into cavity recesses using a "wipe-in" technique. Fill to slight excess. A matrix can be used to give anatomical form.

Once the hardening process begins, the materials should not be disturbed. The material hardens and the matrix can be removed approximately two minutes 30 seconds (37°C or 98°F) after placement.

9. Contouring and Finishing:

DELAY CONTOURING AND FINISHING FOR A MINIMUM OF FIVE MINUTES AFTER MATRIX REMOVAL.

Contour and polish restorations using Sof-Lex™ discs and strip systems.

ADDITIONAL NOTES

1. Sensitivity – some patients may experience transitory postoperative sensitivity. The risk of sensitivity can be minimized by the following measures:

a. Remove minimal tooth structure.

b. Use proper isolation. Use of a rubber dam is highly recommended.

c. Adequate pulp protection. Use a glass ionomer cavity liner/ base on appropriate dentinal surfaces.

d. Adjust occlusion carefully. Check for hyper-occlusion, particularly in lateral excursion contacts.

STORAGE AND USE

1. Do not cross-contaminate contents of jars. Use one end of the disposable mixing spatulas for removing paste A and opposite end for paste B.

2. Do not store materials in proximity to eugenol-containing products.

3. This system should not be exposed to elevated temperatures.

4. Shelf life at room temperature is 18 months. See outer package for expiry date.

5. The system is designed to be used at room temperature of approximately 21-24°C or 70-75°F.

WARRANTY

3M will replace product that is proved to be defective. 3M does not accept liability for any loss or damage, direct or consequential, arising out of the use of or the inability to use these products. Before using, the user should determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.

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