

## Adper™ Prompt™ Self-Etch Adhesive

### Preparation:

Clean unprepared tooth structure with a rubber cup and cleaning paste free of fluoride and oil; rinse thoroughly with water spray and dry.

- Hold vial in a vertical orientation as shown.
- Dispense one drop from **solution A** into the disposable mixing well.
- **Note:** holding the vial vertically for a few seconds before dispensing allows trapped air to escape out of the vial tip.




- Hold vial in a vertical orientation as shown.
- Dispense one drop from **solution B** into the disposable mixing well.
- **Note: to ensure optimum drop ratios, keep the vials vertical when dispensing.**



- Mix the solution aggressively for 5 seconds with the disposable mixing brush.



## Adper™ Prompt™ Self-Etch Adhesive

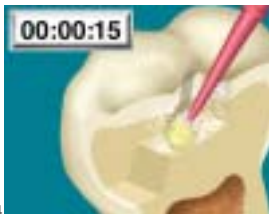

- Apply adhesive to the entire surface of the cavity, rubbing in the solution with moderate finger pressure for 15 seconds.
- **Do not shorten this time. Proper bonding will not occur if the solution is simply applied and dispersed.**



- Use a gentle stream of air to **thoroughly** dry the adhesive to a thin film.



- Apply a second coat of adhesive. This coat does not need to be massaged into the surface.
- Again, use a gentle stream of air to thoroughly dry the adhesive to a thin film.



- Light cure for ten seconds (3 second exposure with a plasma arc lamp).

2 of 2