3M ESPE
Impresept™
Immersion Bath II
 Tauchbad II
 Bac de décontamination II
 Bagno di Immersione II
 Recipiente de inmersión II
 Bacia de Imersão II
 Dompelbad II
 Λουτρό Εμβύθισης II
 Immersionsbad II
 Uputushaude II
 Væskebad II
 Immersjonsbad II

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St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249

3M, ESPE and Impresept are trademarks of 3M or 3M ESPE AG. © 3M 2002. All rights reserved.

ENGLISH

Product Description

Up to six trays can be placed in the Impresept Immersion Bath II, manufactured for 3M ESPE.

Instructions for Use should not be discarded for the duration of product use.

Areas of Use

Immersion bath for immersion disinfectants

Technical Data

Length: 325 mm

Width: 265 mm

Height: 155 mm

Adding Disinfectant

- Impresept disinfectant, manufactured for 3M ESPE, causes brown spots on clothing. For this reason, wear protective gloves and protective clothing when working with Impresept. When using disinfectants from other manufacturers, follow the manufacturer’s instructions.
- Pour undiluted ready-to-use disinfectant solution into the dipping bath until it reaches the optimal fill level (see the markings on the outside front of the bath).
- Place the drain basket into the bath.
- Close the cover so that the disinfectant’s active ingredients do not evaporate.

Disinfecting

- Place the impression trays to be disinfected in the drain basket. **Note: The trays must be completely immersed and covered.** Close the cover.
- When Impresept is used, the impressions must be removed after 10 minutes, so a timer should be used. For other disinfectants from other manufacturers, follow instructions for use. The recommended time for dipping the trays should not be exceeded because the impressions may swell to a greater or lesser degree depending on the material and lose their dimensional stability.

- Hang the basket on the upper edge of the bath and allow the impressions to drain.
- Next rinse the impressions under running water while wearing protective gloves.
- Close the cover of the dipping bath.
- The disinfectant must be changed weekly to ensure its effectiveness.

Cleaning and Disinfecting the Dipping Bath

- Use a soft cloth and if necessary a gentle cleaner to clean the outside of the bath. Do not use solvents or scouring agents which will damage the plastic parts of the bath. Use a brush to clean the inside.
- To disinfect the outside of the bath, use a commercial spray disinfectant. Spray it on a cloth, and use the cloth to wipe the bath.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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