

Adper™ Scotchbond™ Multi-Purpose and Multi-Purpose Plus Adhesive Systems

Preparation:

- Isolate and clean the tooth. Use a diamond to remove weakened porcelain. Roughen alloy surfaces. Bevel the margin and remove 1 mm of surface glaze beyond the margin.



Etch:

- Apply Scotchbond™ etchant to porcelain and alloy—wait 15 seconds.
- Rinse.
- Dry.



Silane Prime:

- Apply RelyX™ Ceramic Primer to porcelain and alloy surfaces.
- Dry.



Bond:

- Apply adhesive to the **silane primed** surfaces.
- Light-cure for 10 seconds.



Composite:

- Complete the repair with the Filtek™ restorative of your choice.

