Instructions for Use  
Mode d'emploi  
Informacion de uso

**3M ESPE AG**  
Dental Products  
D-82229 Seefeld – Germany

**3M ESPE**  
Dental Products  
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE, Garant, Protemp and RelyX are trademarks of 3M or 3M ESPE AG.  
© 3M 2006. All rights reserved.

## ENGLISH

### Product Description

Protemp™ 3 Garant™ manufactured by 3M ESPE is a composite material for the fabrication of temporary restoration at the chairside. Based on multi-functional methacrylate esters, this two-component system has restorative material-like properties and is available in several shades: A1, A2, A3, bleach shade B0,5 and B3. With the Garant™ 2 Dispenser manufactured for 3M ESPE the material can be applied directly from the cartridge without causing voids.

Temporary restorations manufactured with ProtEMP 3 Garant can be built-up and individualized with Protemp™ 3 AddOn composite A1, A2 and A3, manufactured by 3M ESPE. Protemp 3 Garant and flowable composites can also be used.

For details on the products, Garant 2 Dispenser and RelyX™ Temp NE manufactured by 3M ESPE please refer to the corresponding Instructions for Use.

☞ Instructions for Use should not be discarded for the duration of product use.

### Application Areas

- Fabrication of temporary crowns, bridges, inlays, onlays and veneers.
- Crown lining material for 3M ESPE Prefabricated Temporary Crowns.

### Preparatory Work

- Take an alginate or silicone impression from the intact row of teeth.
  - Instead of an impression, a laboratory-made vacuum-formed template or, for single crowns, a preformed crown can be used for shaping.
- To enhance the strength of the temporary restoration, remove interproximal grooves in the impression. If required due to tight spacing conditions, enlarge the relevant sites by cutting.
- For easy replacement in the mouth, relieve undercuts.
- With missing teeth or gaps in the molar area, cut a groove in the impression to produce a stable, bar-shaped connection in the temporary restoration.
- If required, prior to taking the impression close any gaps between front teeth with denture teeth acting as spacers; interlock multiple denture teeth with wax.

### Times

At room temperature (73°F) and 50% relative humidity the following time periods are available for processing:

min:sec	min:sec	min:sec	min:sec	min:sec
00:00	00:50	01:35	02:30	05:00
	Fill the impression and reposition in the mouth	Setting in the mouth	Withdrawal from the mouth	Complete setting
				Finishing

### Dispensing and Mixing

The pastes are dispensed and auto-mixed from the Garant 2 dispenser. Use the enclosed blue Garant 2 Mixing tips only!

Operational check for new cartridges:

- Remove and discard the cartridge cap.
- Check whether both cartridge openings are free from obstruction and remove obstructing material, if any, with a suitable instrument.
- Extrude a small quantity of base paste and catalyst each until both pastes emerge uniformly.

### Application

Use gentle and consistent pressure to dispense material from the mixing tip. Increasing the pressure does not accelerate the flow! The flow of material is interrupted as soon as the pressure on the handle is reduced. Material that has set inside the mixing tip should not be extruded by force, as this may damage both the cartridge and the mixing tip.

- First, install a new mixing tip.
- Prior to each application, extrude a small amount (pea-sized) material from the mixing tip onto a mixing pad and discard this material.
- Subsequently, load the dried impression or vacuum-formed template in the relevant spaces from bottom up.
- Reposition the impression or vacuum-formed template in the mouth.
- The material attains a hard-elastic consistency approx. 1 min 35 sec after the onset of mixing. The material and the impression or vacuum-formed template must be removed from the mouth within approx. 2 min 30 sec after the onset of mixing.
- Store cartridge with used mixing tip attached until next use.

### Finishing

Do not breathe the polishing dust; use suitable mouth protective device or aspiration!

- Finish the temporary restoration once the material is completely cured (not earlier than 5 minutes after the onset of mixing) using fine carbide burs. Polish if desired.
  - Remove oxygen-inhibited layer on the surface of the restoration with organic solvents such as ethanol.

### Individualization / Shape Correction

Using Protemp 3 AddOn and Protemp 3 Garant for custom-shaping and shape correction there usually is no need for a bonding agent. If you intend to use other products, please refer to the pertinent manufacturer’s Instructions for Use.

### Compatible Products

- Protemp 3 AddOn composites
- Flowable composites
  - Please comply with the corresponding Instructions for Use!
- Protemp 3 Garant
  - Processing: please refer to the information provided under “Application” and “Finishing”.

### Application of Protemp 3 AddOn to New Temporary Restorations

In newly manufactured temporary restorations, Protemp 3 AddOn can be applied either to the finished surface or the unprocessed, but clean, oxygen inhibition layer. Bonding is excellent in either case.

- Any contamination – e.g. from saliva or polishing dust – should be removed with alcohol from temporary restorations with inhibition layers and finished temporary restorations, which should then be dried with a stream of air.

Alternatively: If the inhibition layer or the finished restoration is clean, proceed as described in the next step.

- For reasons of hygiene, dispense Protemp 3 AddOn composite onto a mixing pad.
- Apply the composite in 1 mm increments to the temporary restoration.
- Light-cure each increment for 20 sec.
- Finish and polish the restoration with carbide burs and polish with polishing paste.

### Application of Flowable Composites to New Temporary Restorations

- Remove the oxygen-inhibited layer from the restoration.
- Thoroughly roughen the surface of the temporary restoration with rotary instruments.
- Remove any contamination – e.g. from saliva or polishing dust – with alcohol, then dry the restoration with a stream of air.
- Place flowable composite following the manufacturer’s instructions.

### Application of Protemp 3 AddOn to Temporary Restorations that were in Use

- Thoroughly roughen the surface of the temporary restoration with rotary instruments.
- Remove any contamination – e.g. from saliva or polishing dust – with alcohol, then dry the restoration with a stream of air.
- For reasons of hygiene, dispense Protemp 3 AddOn composite onto a mixing pad.
- Apply the composite in 1 mm increments to the temporary restoration.
- Light-cure each increment for 20 sec.
- Finish and polish the restoration with carbide burs and polish with polishing paste.

### Application of Flowable Composites to Temporary Restorations that were in Use

- Thoroughly roughen the surface of the temporary restoration with rotary instruments.
- Remove any contamination – e.g. from saliva or polishing dust – with alcohol, then dry the restoration with a stream of air.
- Place flowable composite following the manufacturer’s instructions.

### Technique for Lining Prefabricated Temporary Crowns

- Trim, contour and crimp temporary crown to fit crown preparation.
- When a satisfactory fit has been achieved, dispense the Protemp 3 Garant material into the prefabricated crown. Fill the crown slightly over half with material. **Do not overfill.**
  - Note: When using a polycarbonate crown, it is recommended that a thin layer of methylmethacrylate be brushed on the inner surface of the crown and air dried for better adhesion.**
- Seat crown into place and guide patient into occlusion for posterior crowns.
- Remove crown from prepared tooth 1:35 min :sec after dispensing. If desired, it may remain in the mouth for as long as 5 minutes.
- Trim excess material at the margins.
- Reseat crown on the tooth preparation to verify proper fit.
- Cement crown with a temporary cement. Refer to the information provided under “cementation”.

### Cementation

Eugenol-containing temporary cements may impair the setting or bonding of the permanent cement when seating the final restoration.

- Insert the temporary restoration with a commercial temporary cement, e.g. RelyX Temp NE.

### Cementation of Veneers

For improved esthetics, the Protemp 3 Garant temporary veneers can be temporarily cemented using the “bonded button technique” with a permanent resin cement.

- Spot etch the center of the facial surface approximately 1–2 mm for 20 seconds with standard phosphoric acid etching gel. Caution: Etching beyond the 1–2 mm area will cause difficulty in removal of the temporary veneer.
- Blot excess water leaving the tooth moist.
- Seat the temporary veneer with a resin cement, e.g. RelyX™ Veneer Cement manufactured by 3M ESPE, making sure that the composite is not bonded to the entire surface, otherwise removal will be impeded.
- Lightcure and clean up the resin cement according to the manufacturer’s recommendations.

### Removing the Temporary Restoration

- Remove the restoration using an appropriate instrument.
- Remove residual cement.
- Bond the final restoration in the usual manner.

### After Cementation

Thoroughly examine and explore the sulcus and surrounding dentition. Remove any residuals of the temporary material or cement from the mouth.

### Repair

- If a restoration breaks shortly after fabrication, repair it using a fresh mix of Protemp 3 Garant.
- Breakage of older or worn restorations: Roughen the fractured surfaces and adjacent areas, place undercuts and re-assemble and bond the pieces in the proper orientation using a fresh mix of Protemp 3 Garant.
- Once the material is fully set, finish as usual.

### Cleaning

- Remove paste that has not set using an alcohol-soaked cloth.
- The Dispenser handle and plunger can be autoclaved up to a temperature of 135°C/275°F. Disassemble the Dispenser before autoclaving.
- Glutaraldehyde-based solutions can be used for disinfection.

### Precautionary Information

#### For Patients:

- Avoid use of this product in patients with known acrylate allergies.
- This product contains substances that may cause an allergic reaction by skin contact in certain individuals.
- If prolonged contact with oral soft tissue occurs, flush with large amounts of water.
- If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

#### For Dental Personnel:

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product.
- If skin contact occurs, wash skin with soap and water.
- Use of protective gloves and a no-touch technique is recommended.
- Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove.
- If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

### Storage and Stability

Store the product at 15–25°C/59–77°F.

Do not store in the refrigerator.

Do not use after the expiration date.

### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

### Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

### Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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