Svenska

Beskrivning:

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English

Description:

The **Fluid Resistant/Ear Loop/Tie-On/Triple Layer** Molded Face Masks are a no fiberglass, single-use mask.

Intended Use:

These products are intended for infection control practices to minimize contamination caused by exhaled microorganisms and reduce the potential exposure of the wearer to blood and body fluids.

The Triple Layer Mask may be used by health care professionals desiring higher filtration efficiency than standard surgical masks provide.

Contraindications:

These devices are not designed, sold, or intended for use except as indicated.

Use of these products does not provide respiratory protection. OSHA has not set an acceptable exposure level for biohazards nor approved respirators for biohazards.

Warning:

The Fluid Resistant Face Masks and Triple Layer Molded Face Masks have components that contain natural rubber latex which may cause allergic reactions in some individuals.

These products are intended for infection control practices in health care. When properly worn, they reduce potential exposure of the wearer to blood and body fluids but do not eliminate the risk of contracting any disease or infection. Change immediately if contaminated with blood or body fluid.

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