ENGLISH

3M Masking Agent Instructions

3M Masking Agent has been designed for those restorations where greater masking is required than can be obtained with a 3M restorative shade alone. It is indicated for use as an undercoating to mask enamel imperfections and discolorations prior to veneering enamel surfaces. It is also used to mask pins and exposed metals as in porcelain repair and cosmetic covering of mesial amalgams.

To enhance marginal sealing and provide retention, 3M Masking Agent should be used with the acid etch technique and application of a Scotchbond adhesive.

Shades available: Universal: B3; Dark Yellow: A4; Gray: D3; White.

Precautions for Dental Personnel and Patients: Masking Agent Precautions

Masking Agent contains BIS-GMA and TEGDMA Resins. A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resins should be avoided. If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If skin contact occurs, wash skin with soap and water.

Step 1 - Tooth Preparation

- Prophylax the enamel surface(s) to be etched using pumice and water. Do not use prophylax pastes containing oils or fluorides.
- Rinse thoroughly with water and dry.
- Choose the appropriate shade that will blend with the surrounding tooth structure.

Step 2 - Etching

Use a matrix strip to protect adjacent tooth surface(s) during etching. Etch according to the instructions of the Scotchbond adhesive product that is used.

- If metal surfaces or fractured porcelain are to be masked, these surfaces are also treated with the etchant for cleaning. Do not contaminate from this point on. If contamination occurs, re-etch.

Dry with oil-free, moisture-free air.

Step 3 - Adhesive Application

Apply and cure the adhesive according to the instructions of the Scotchbond adhesive used. In the case of porcelain repair, Scotchbond™ Ceramic Primer should be placed on the porcelain surface and exposed metal prior to the placement of the adhesive.

Step 4 - Application of 3M Masking Agent

- Dispense the required amount of 3M Masking Agent onto pad and immediately replace the syringe cap.
- Place the selected shade of 3M Masking Agent immediately and brush a thin layer (0.25 - 0.5 mm) over the areas to be masked. Cure 20 seconds. Additional layers of Masking Agent may be added to mask about 75% of the stain (cure each layer 20 seconds).

If tints are to be used, a thin coat of the appropriate shade is placed over the Masking Agent and cured 20 seconds. Refer to the appropriate instructions.

Step 5 - Completion of Restoration

- Complete the restoration with an appropriate shade of restorative. See appropriate instructions.
- Finish the restoration with the Sof-Lex™ Disc System.

Storage and Use

1. Do not expose materials to elevated temperature or intense light.
2. Do not store materials in proximity to eugenol-containing products.
3. This system is designed to be used at room temperature of approximately 21- 24° C or 70- 75° F.
4. Shelf life at room temperature is 36 months. See outer package for expiry date.

Warranty

3M will replace such product that is proved to be defective. 3M does not accept liability for any loss or damage, direct or consequential, arising out of the use of or the inability to use these products. Before using, the user shall determine the suitability of the product for its intended use and user assumes all risk and liability whatsoever in connection therewith.

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